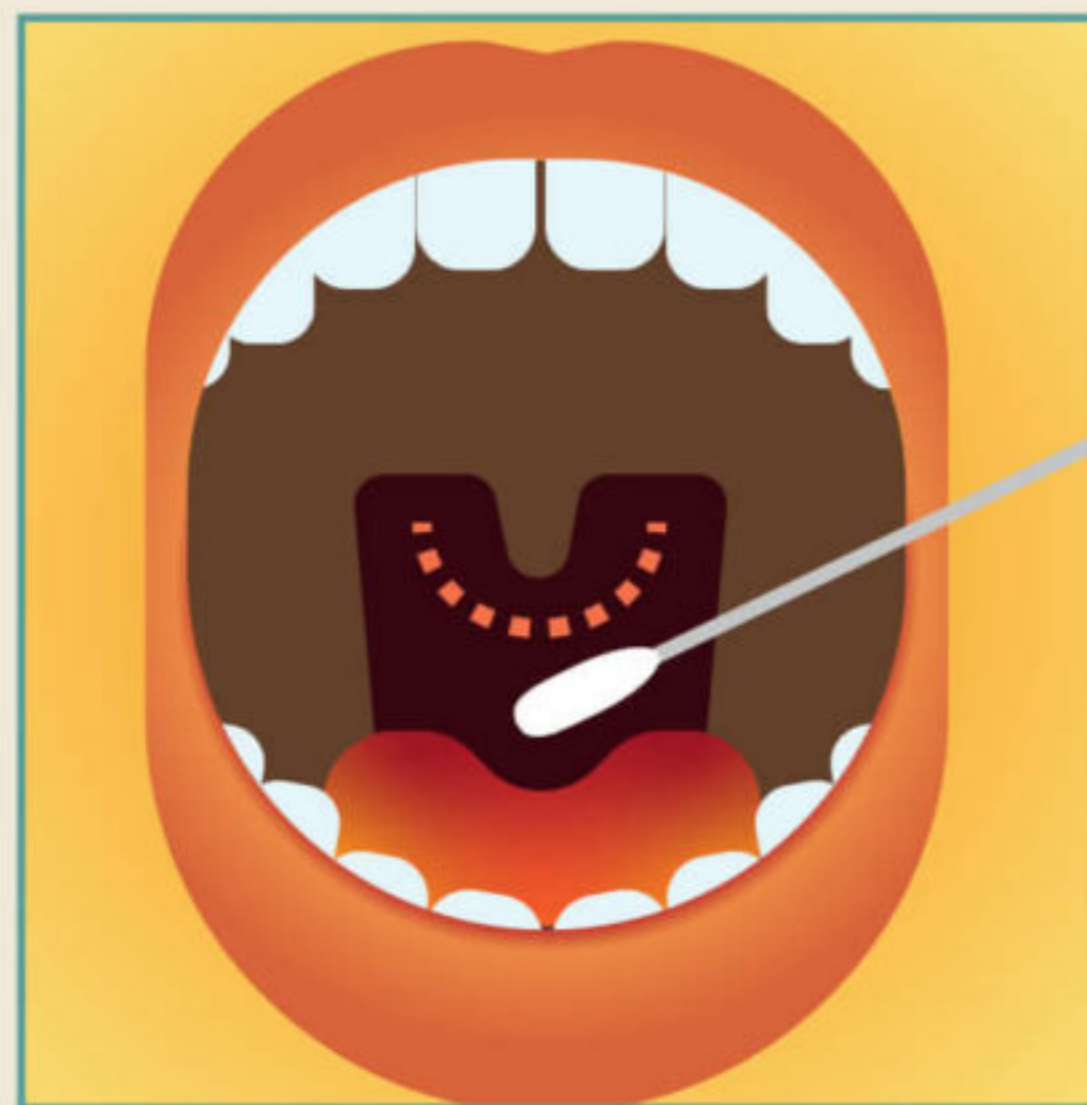
Instrukcja pobierania materiału do badania. Instructions for taking material for testing.

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Wash your hands. Swabs are labeled. Remove the sterile swab without touching it or any surfaces and follow step 2.

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
In front of the mirror, vigorously rub the back of the throat several times with the swab, without touching the surface of the mouth. A gag reflex may appear.

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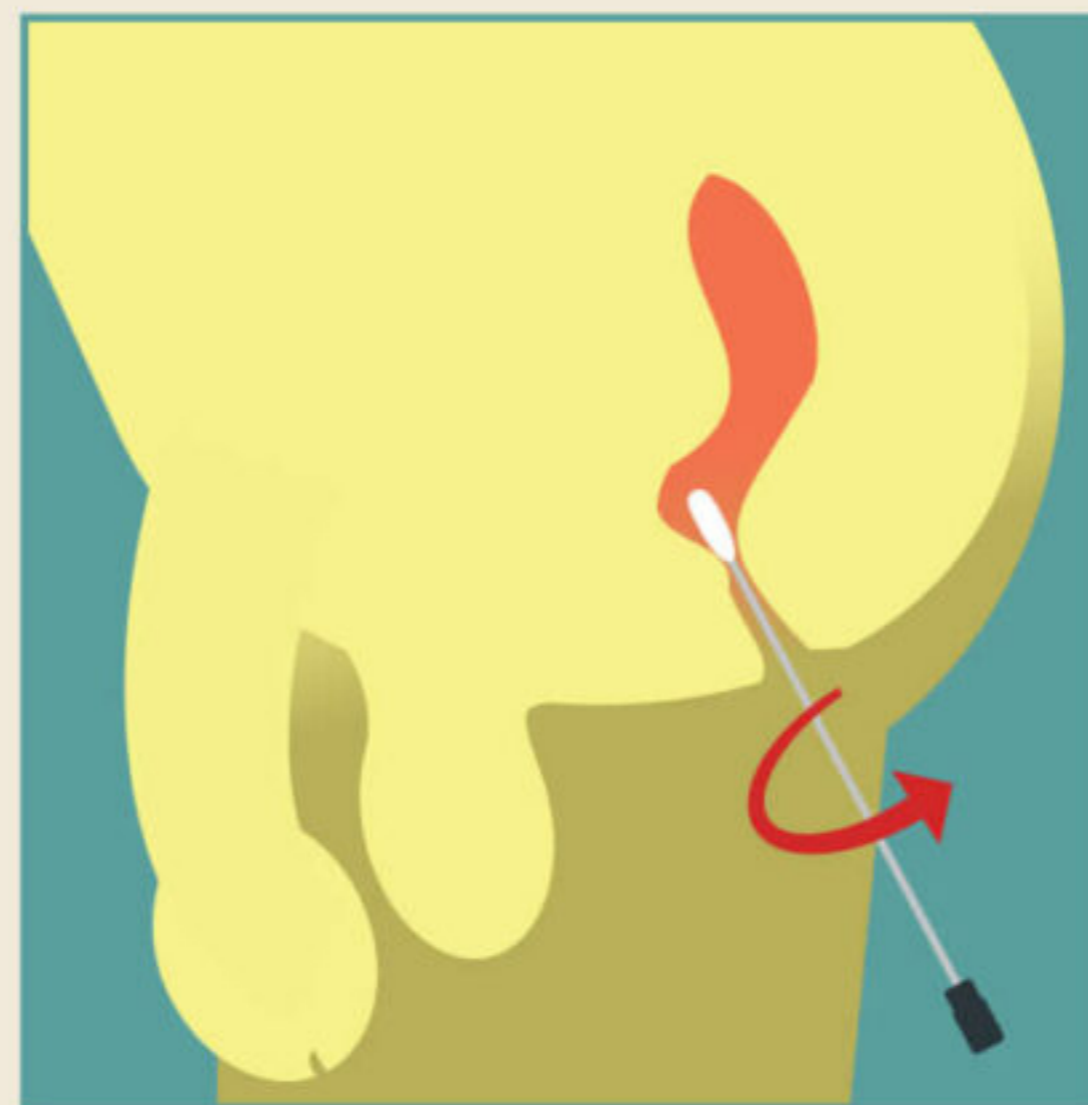
Relax the anal sphincter muscles and gently insert the swab about 5 cm. Be careful not to break it. Twist the swab 5 times. There should be a trace of stool on the swab.

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
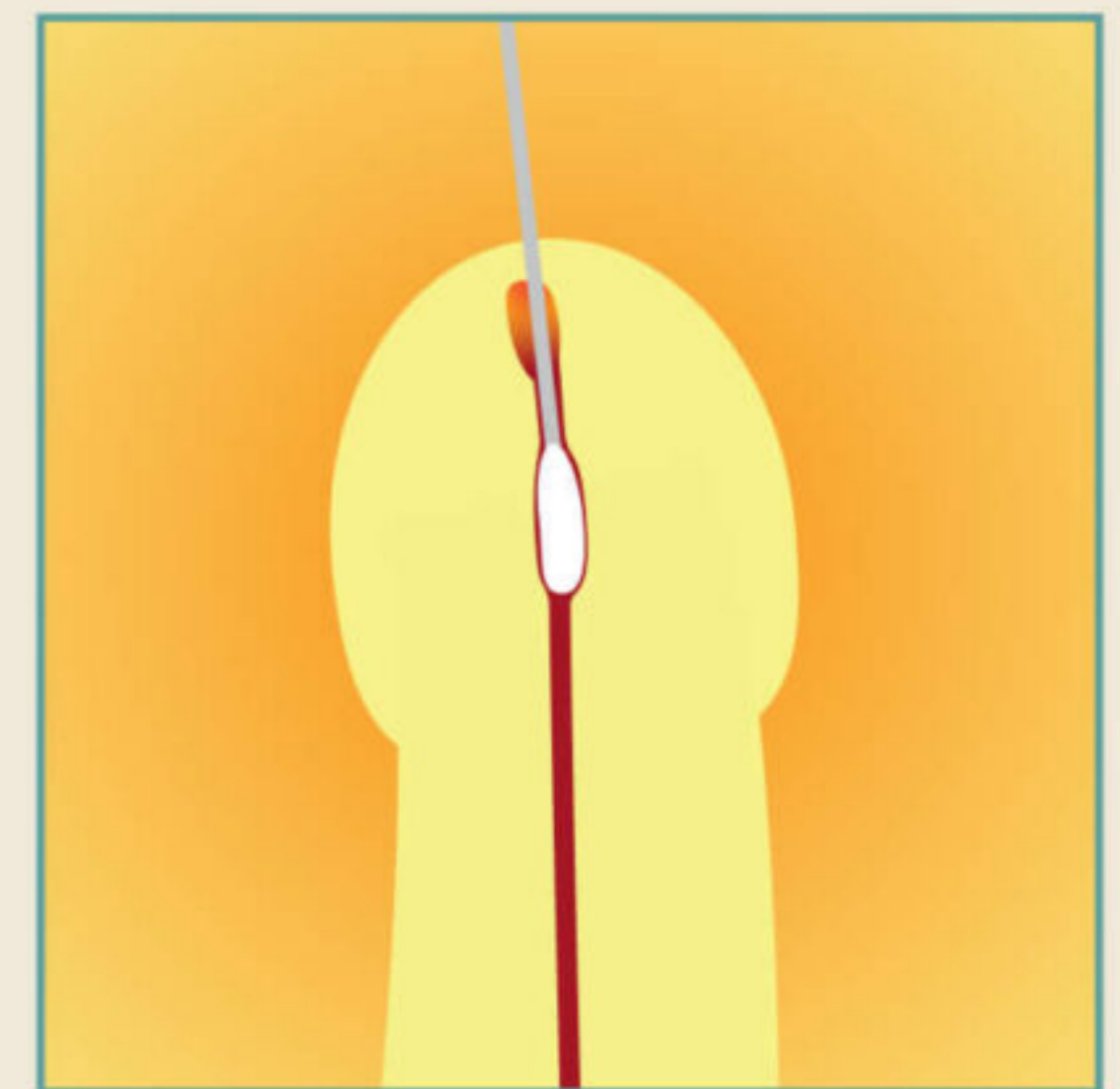
Insert the swab 1 cm deep into the urethra and hold it there for 5 seconds. The test may be uncomfortable.

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Insert the swab(s) into the transport tube with the appropriate marking for the collection area. Brake the swab at the narrowing. Twist the cap.

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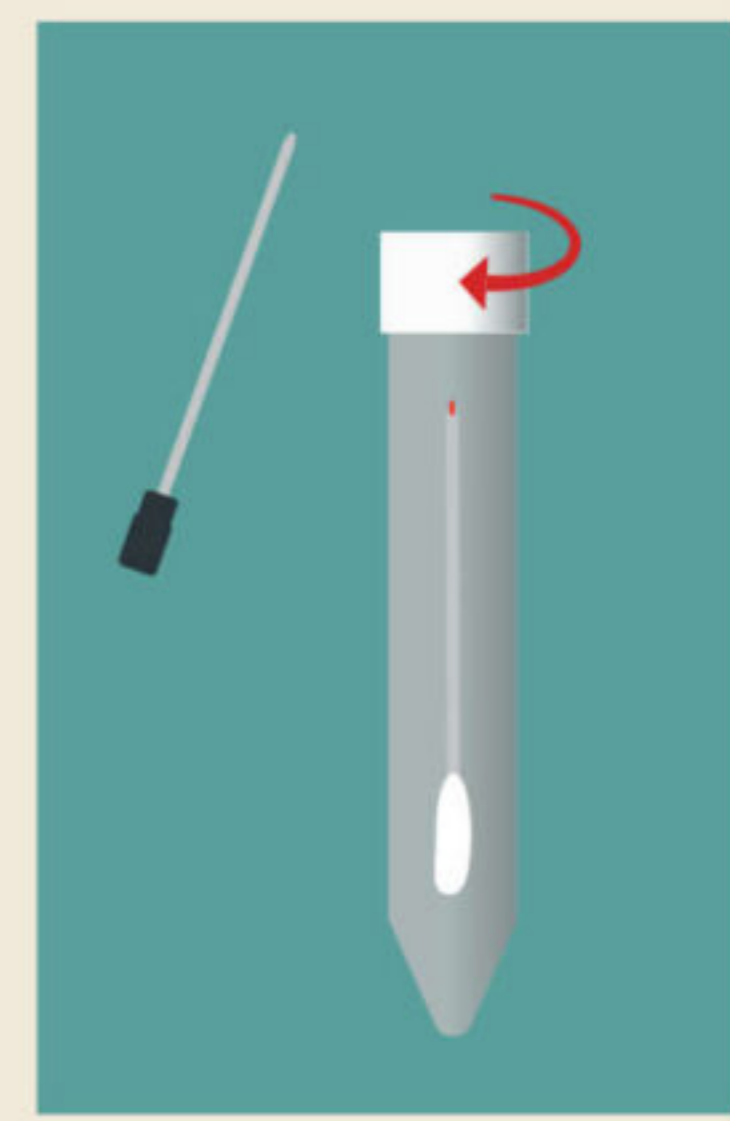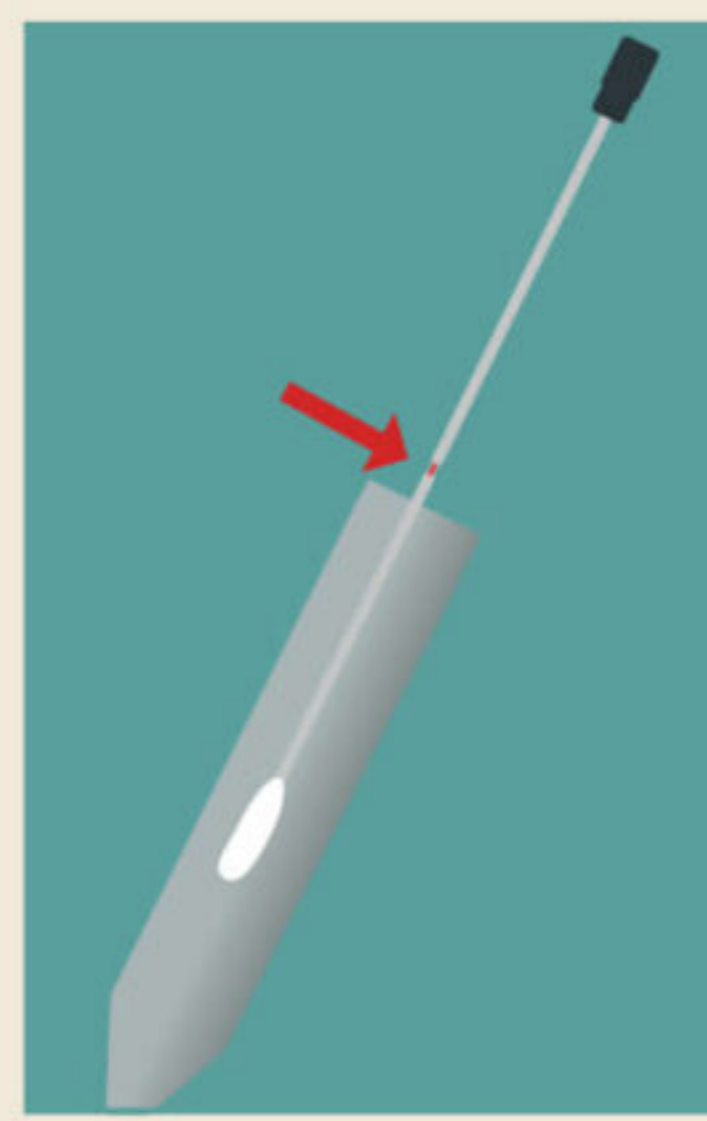For swabs - put the return tube(s) in the same shipping box in which you received the swabs. Leave the enclosed referral inside the box. If collecting a urine sample at the same time, put the zipper bag with the packaged sample along with the cooling cartridge into the foil transport envelope (poly mailer). Put the box with the packed swabs into the same envelope.

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
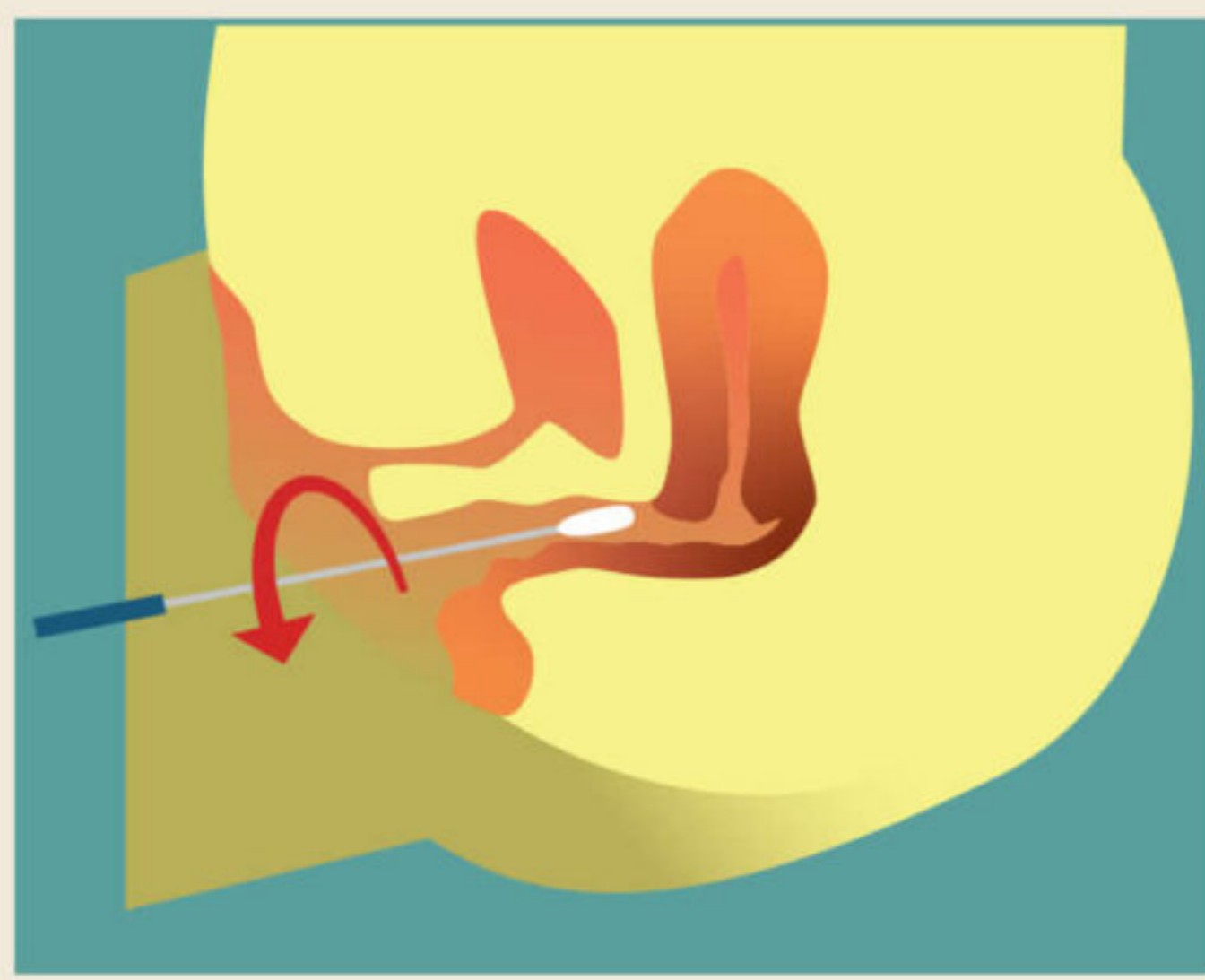
Perform the examination 7 days after the use of vaginal drugs, 24 hours after sexual contact and / or gynecological examination or vaginal ultrasound. While standing or sitting on the toilet, open the labia with your fingers and insert the swab into the vagina about 3 cm. Twist the swab once. Count to 10 and take out the swab.

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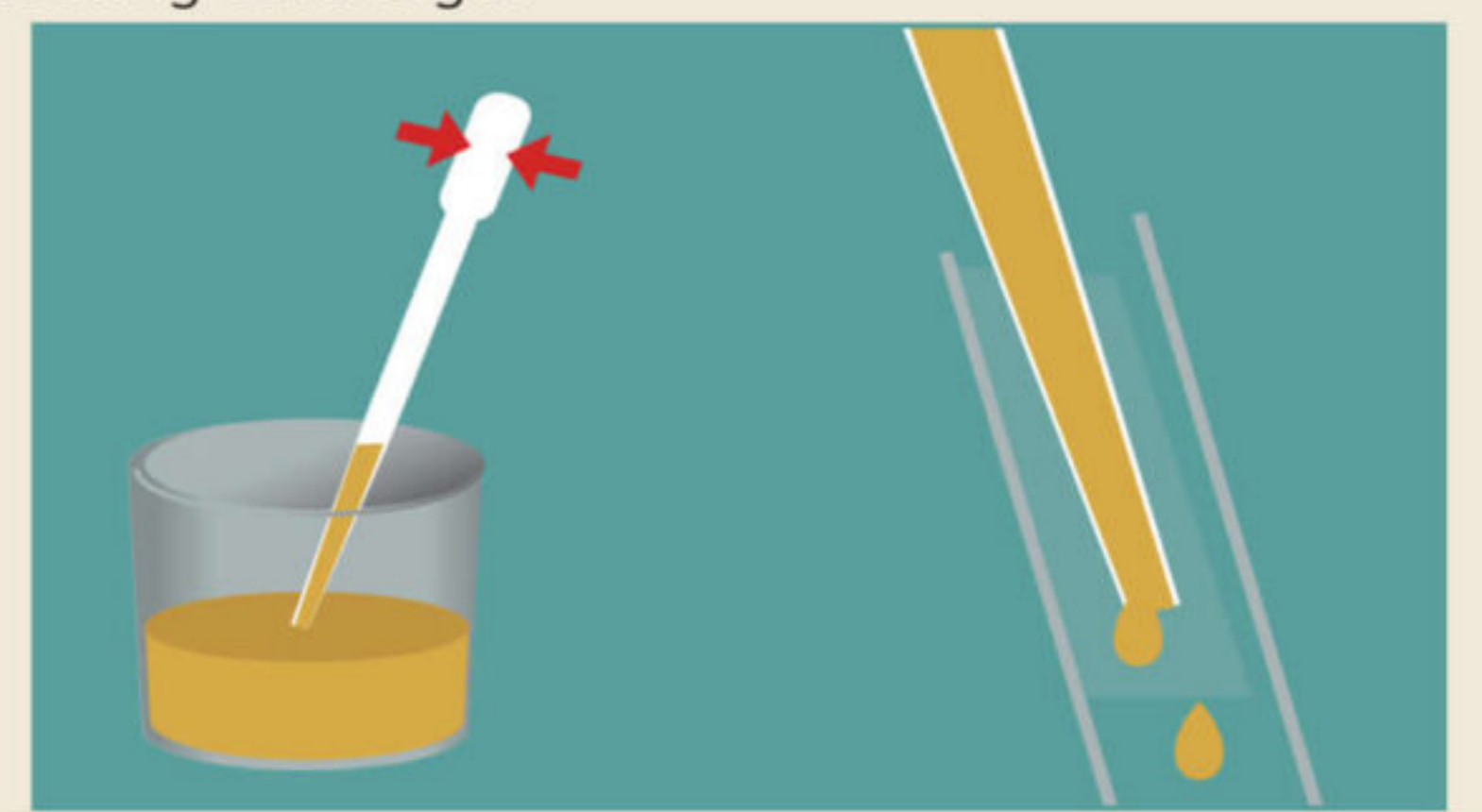Put the cooling cartridge in the freezer overnight. The next day, urinate from the first (morning) stream into a sterile container (included in the kit). Fill about half of the container. Transfer the urine with the enclosed pipette into a urine transport tube (about 2 ml). Tightly close the tube. Place the tube in a zipper pouch with an adsorbent cartridge. Pack the zipper bag with the filled tube in a larger zipper bag into which you also put the frozen cooling cartridge.

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